

Always advancing, never stopping....
Because **Every Test is a Life™**

The first COVID-19 Total Antibody Test available for high-throughput, random access, fully automated instruments

Authorized by the FDA for Emergency Use

To help identify the immune response in individuals who are currently or were previously exposed to or infected with SARS-CoV-2.¹



Aids in the diagnosis of SARS-CoV-2



Provides rapid and wide-scale access to testing



Measures your patients immune response

Results You Can Trust:

Clinical Specificity:



VITROS® Total Antibody Test targets the S protein unique to SARS-CoV-2, minimizing the risk of cross-reactivity with other strains of the coronavirus.

Assay Accuracy & Reproducibility



Enabled by Ortho's VITROS® MicroWell Technology



Enhanced chemiluminescence detection technology




Broad dynamic range



Disposable Versatips eliminate the risk of carryover



Liquid ready to use reagents

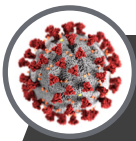

SPECIFICATIONS

Assay: VITROS® Anti-SARS-CoV-2 Total
Test Type: Immunometric
Systems: VITROS® XT 7600, 5600, 3600
Incubation Time: 37 min
Time to First Result: 48 min
Reaction Sample Volume: 80 uL
Calibration Stability: 28 days or lot change
On-board Stability: ≤ 8 weeks

CPT Code: 6199922

Interpretation of Results:

Result (S/C)	< 1.00	≥ 1.00
Result Text	Non-reactive	Reactive

How to Get Started:



1 Order reagent pack, calibrator, and control material.
 VITROS Anti-SARS-CoV-2 Total:
 Reagent pack (619 9922)
 Calibrator (619 9923)
 Controls (619 9924)

2 Update LIS
LIS Assay Code: 89
Report Name: CoV2T
Body Fluid Key: 5 Serum, 6 Plasma
LOINC: 94547-7

3 Validate
COLA recommended validation procedure: Run 2 negative and 2 positive controls in duplicate on 2 successive days.



Ortho manufactures all reagents, calibrators, and QC in-house, ensuring continuity of supply to meet the demands of today and tomorrow.

Ordering Information

Assay:	Anti-SARS-CoV-2 Total
Reagent Catalog #:	6199922
Calibrator Catalog #:	6199923
Control Catalog #:	6199924
LIS Assay Code:	89
Report Name:	CoV2T
Body Fluid Key:	5 Serum, 6 Plasma
LOINC:	94547-7

Please contact your local laboratory specialist if you have any questions or concerns.